




TERRAN

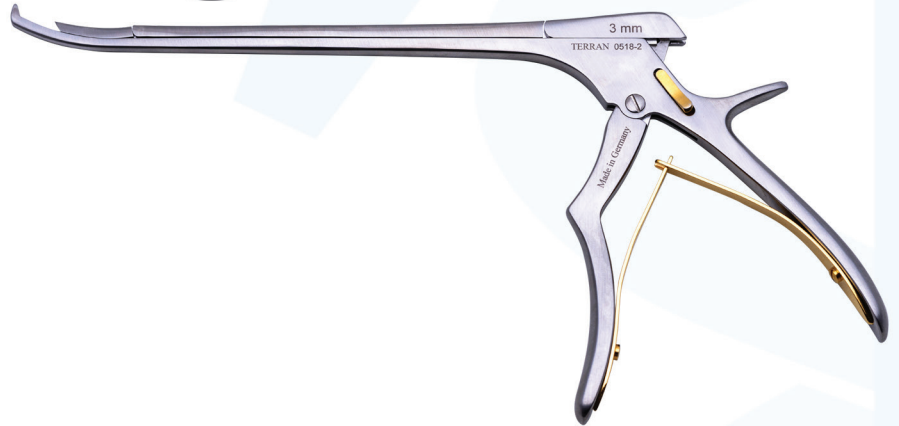

KERRISON RONGEURS

BENEFITS







1. Improve surgical outcome by reducing cross contamination
2. Easy access to clean the inside of the instrument with no mismatched parts
3. Ultra-thin foot plate allowing access to all anatomical areas
4. Simple to open and close for cleaning
5. Designed to reduce sticking and provide smooth cutting actuation
6. Solid ejector and improved cutting resulting in reduced hand fatigue





Ferris - Smith Cleanable Kerrison

  40° Forward Cutting	Shaft Length 20 cm
--	---







					
1mm 45-920.01DTR	2mm 45-920.102DTR	3mm 45-920.103DTR	4mm 45-920.104DTR	5mm 45-920.105DTR	6mm 45-920.106DTR

Note: Different lengths, bayonet shafts, and 90° bite angle available.



Cloward Cleanable Kerrison

 <p>40° Forward Cutting</p>	<p>Shaft Length</p> <p>20 cm</p>
--	---

					
<p>1mm</p> <p>45-920-11DTR</p>	<p>2mm</p> <p>45-920-112DTR</p>	<p>3mm</p> <p>45-920-113DTR</p>	<p>4mm</p> <p>45-920-114DTR</p>	<p>5mm</p> <p>45-920-115DTR</p>	<p>6mm</p> <p>45-920-116DTR</p>



Love Cleanable Kerrison	
<p>40° Forward Cutting</p>	<p>Shaft Length 20 cm</p>

1mm 45-420.11TR	2mm 45-420.102TR	3mm 45-420.103TR	4mm 45-420.104TR	5mm 45-420.105TR	6mm 45-420.106TR



Ferris - Smith Cleanable Foraminotomy Kerrison	
<p>Hinged Tip Curved Up 70°</p>	<p>Shaft Length 20 cm</p>


2mm 45-950.02DTR	3mm 45-950.03DTR	4mm 45-950.04DTR

Cleanable Kerrison Rongeurs Instructions for use

How to open and close


Open

Step 1 Press gold button to open as shown below.

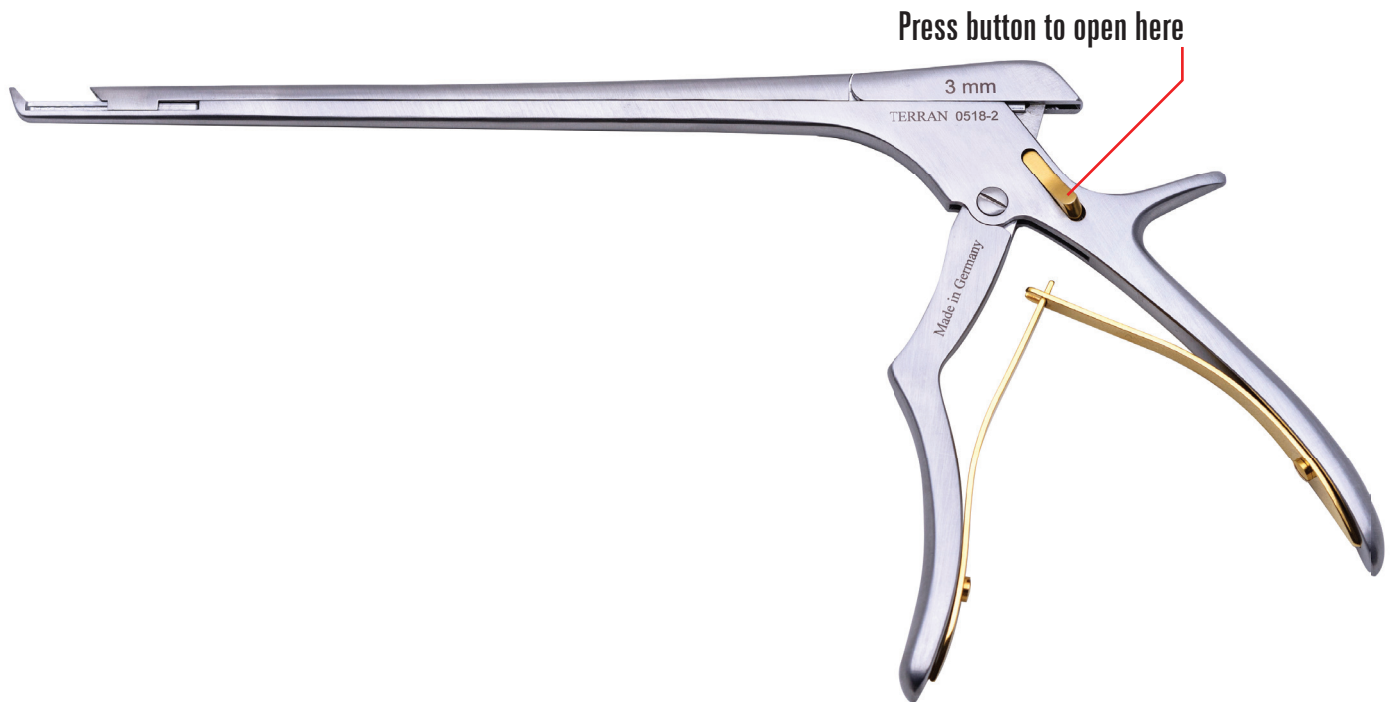

Step 2 Clean to remove all foreign debris from the instrument. Lubricate "L" locations as shown below.

Remove all foreign debris from Slots

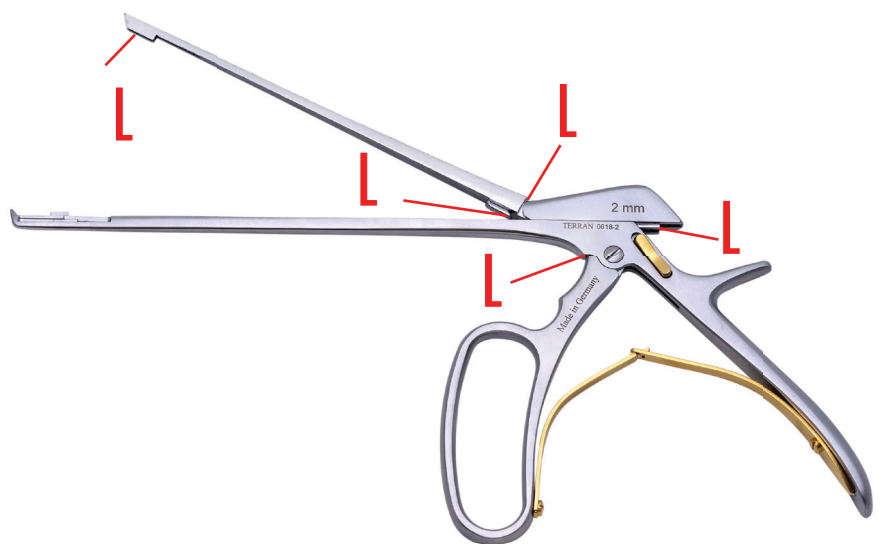


Close

Step 3 First hold top and bottom jaws together "A". Then squeeze handle "B" as shown below.

If the instrument jaw will not close; press the gold button (step 1), then repeat step 3.

Caution: make sure your finger is not in the cutting location.

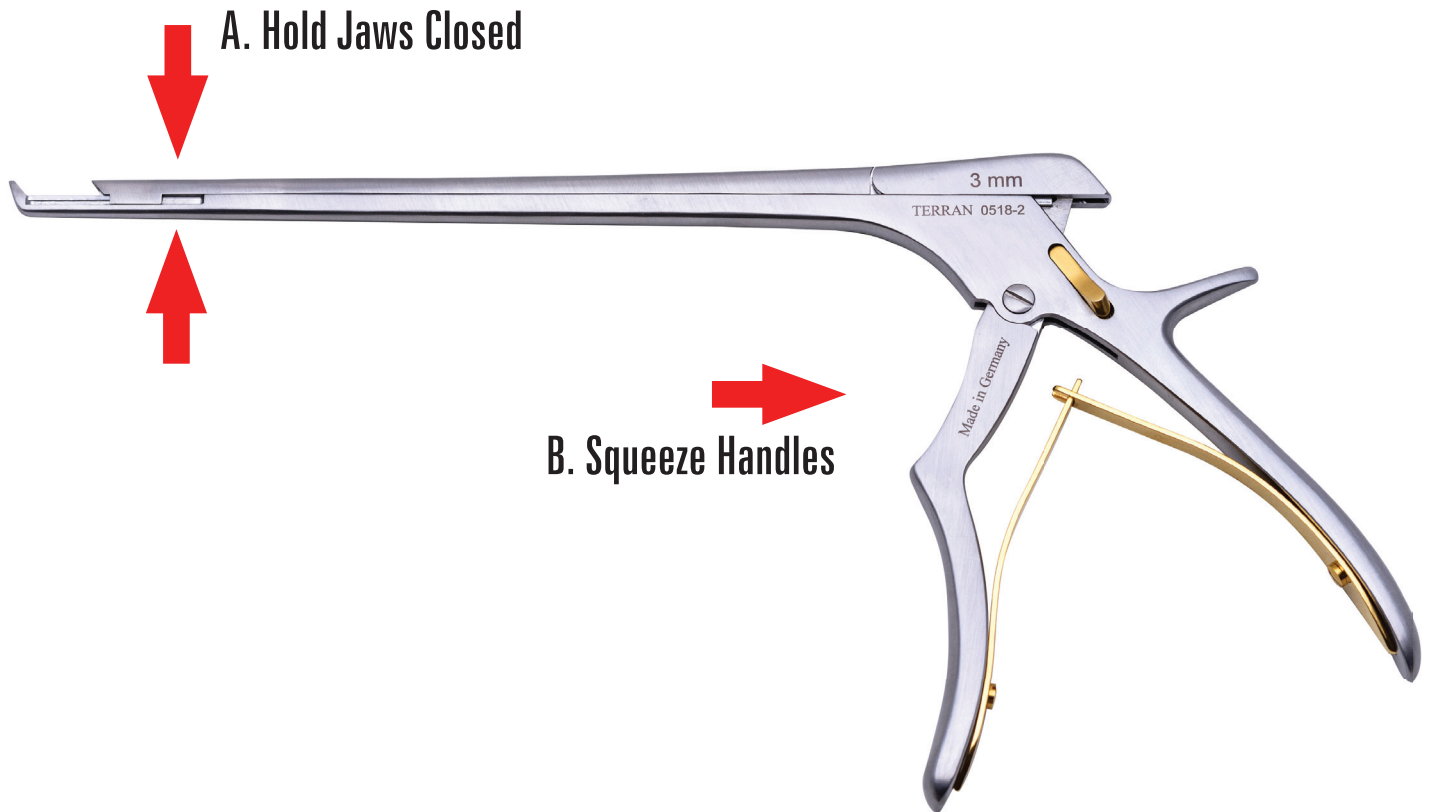

Important Instrument Care:

1. These instruments should only be used as intended.
2. After cleaning and lubricating instrument should be inspected to assure proper function before returning to service.
3. Damaged and bent instruments should not be used.
4. Instrument must be lubricated with AESCULAP STERILIT I (Ref. JG598) lubricant or equivalent.
5. Lubricant should remain in the instrument after Autoclave cycle.
6. Autoclave Cycle: Pre Vac 132° 6 minutes minimum. Drying time 30 minutes minimum or equivalent.



TERRAN provides world-class surgical instruments to meet the requirements of the experts.

Our goal is to ensure customers can safely use our products to improve clinical results.

We start by selecting the top materials suitable for the application and strive to provide a product that exceeds our customers' expectations. Assuring the quality and reliability of the products' performance is important to us. We offer a limited warranty on all our products.

**cs@terranmed.com
1-508-818-8468**

www.terranmed.com

**6 Hunters Run
Franklin, MA 02038**