



TERRAN



FLEXIBLE OSTEOTOME SET



**24221
Osteotome Handle**



**4100
Short Blade
6 mm**



**4200
Short Blade
10 mm**



**4300
Long Blade
6 mm**



**4600
Long Blade
10 mm**



**3200
Cupped Blade
12 mm**



**7500
Curved and Cupped Blade
12mm**



Flexible Osteotome Set

Instructions for use

Load blades into the handle

1. Turn knob counter clockwise to loosen.
Caution to not over loosen knob and break screw.
2. Slide blade into slot as shown
3. Turn knob clockwise to tighten blades
4. Check if the blade is secure in the handle before use

Remove blade from the handle

1. Turn knob counter clockwise to loosen.
Caution to not over loosen knob and break screw.
2. Slide blade out of slot as shown

CAUTION

Appropriate personal protective equipment such as safety goggles and gloves must be worn to protect against hazards that may be encountered while using surgical instruments.

Important Instrument Care:

1. These instruments should only be used as intended.
2. After cleaning instrument should be inspected to assure proper function before returning to service.
3. Damaged and bent instruments should not be used.
4. Autoclave Cycle: Pre Vac 132° 6 minutes minimum. Drying time 30 minutes minimum or equivalent.



TERRAN provides world-class surgical instruments to meet the requirements of the experts.

Our goal is to ensure customers can safely use our products to improve clinical results.

We start by selecting the top materials suitable for the application and strive to provide a product that exceeds our customers' expectations. Assuring the quality and reliability of the products' performance is important to us. We offer a limited warranty on all our products.

**cs@terranmed.com
1-508-818-8468**

www.terranmed.com

**6 Hunters Run
Franklin, MA 02038**